

INDICATIONS FOR USE

EZ-Spray[™] powered atomization devices are intended to deliver either oil or water-based solutions such as topical anesthetics, vasoconstrictors and other nasal or oral medications in an aerosol form.

SPECIFICATIONS

Gas source by SMR cart with a pressure hose. Ensure the pressure is set to 10psi (0.7bar) before starting procedure.

WARNINGS

- Not intended for use as a medicinal inhaler with meter dosing requirements.
- DO NOT REUSE with multiple patients.
- Before use, clinician should inspect the device to be sure all parts and connections are secure.

CAUTIONS

The following are general precautions to be aware of relating to EZ-Spray[™] powered atomization device:

- Do not open case packaging with sharp knives to avoid creating accidental cuts in the individual packaging or damage to device.
- EZ-Spray[™] powered atomization device is non-conductive.
- Federal Law (USA) restricts this device to sale by or on the order of a licensed physician.

INSTRUCTIONS FOR USE

EZ-Spray[™] powered atomization device should only be used the following way. See figure 1:

1. Remove the contents from the package.
2. Unscrew the bottle from the Housing of the device
3. Fill the bottle with choice of topical anesthetics, vasoconstrictors, and/or other nasal or oral medications.
4. Screw the bottle to the housing.
5. Connect tubing to SMR cart pressure hose with open-air cut off.
6. Set the pressure to 10psi (0.7bar). Follow the cart manufacturers instructions for pressure adjustments.
7. Ensure all connections are always secure.
8. Dispose of according to local established protocols after use.

Figure 1.

